

## Talking to patients? A Train

### How to Use EVZIO

#### Visual and voice instructions help guide the way

EVZIO is designed to be easy to use for patients, their family members, and other caregivers who do not have medical training.<sup>1</sup> It contains the Intelliject® Prompt System (IPS™) with visual and voice instructions that help guide the user through the injection process.

#### Administration steps

##### **Pull EVZIO from the outer case.**

For more information about EVZIO, call **1-855-77-EVZIO** (1-855-773-8946).

**Do not** go to Step 2 (do not remove the **red** safety guard) until you are ready to use EVZIO. **If you are not ready to use EVZIO, put it back in the outer case for later use.**

##### **Pull off the red safety guard.**

To reduce the chance of an accidental injection, do not touch the **black** base of the auto-injector, which is where the needle comes out. If an accidental injection happens, get medical help right away.

**Note:** The **red** safety guard is made to fit tightly. **Pull firmly to remove.**

**Do not replace the red safety guard after it is removed.**

**Place the black end against the middle of the patient's outer thigh, through clothing (pants, jeans, etc) if necessary, then press firmly and hold in place for 5 seconds.**



If you give EVZIO to an infant less than 1 year old, pinch the middle of the outer thigh before you give EVZIO and continue to pinch while you give EVZIO.

**Note:** EVZIO makes a distinct sound (click and hiss) when it is pressed against the thigh. This is normal and means that EVZIO is working correctly. Keep EVZIO firmly pressed on the thigh for 5 seconds after you hear the click and hiss sound. The needle will inject and then retract back up into the EVZIO auto-injector and is not visible after use.

**After using EVZIO, the user should immediately seek emergency medical help.**

If symptoms return after an injection with EVZIO, an additional injection using another EVZIO may be needed. Give additional injections using a new EVZIO auto-injector every 2 to 3 minutes and continue to closely watch the person until emergency help is received.

EVZIO cannot be reused. After use, place the auto-injector back into its outer case. Do not replace the **red** safety guard.

## A Trainer is Included for Practice

Each EVZIO prescription comes with a black-and-white Trainer that can be used for practice. Unlike EVZIO, the Trainer:

**Does not have a needle**

**Does not have an expiration date**

**Does not contain medicine or any liquid**

**Can be reused (more than 1000 times)**

To be prepared in an opioid overdose emergency, patients, family members, caregivers, and other individuals who may have to administer EVZIO should practice using the Trainer to become familiar with the injection process. After practicing with the Trainer, the electronic voice system should be reset by:

- Replacing the red safety guard
- Sliding the Trainer all the way back into the outer case

The Trainer should be left in its outer case for at least 5 seconds between each

practice interval to allow the electronic voice system to reset properly. For more information on the Trainer, view the [Trainer Information](#).

## REFILL REMINDER

**Patients should refill their prescription even if only a single EVZIO auto-injector is used to help ensure they always have 2 EVZIO auto-injectors by their side.**

Reference: 1. Data on file. kaleo, Inc.

©2016 kaleo, Inc. All rights reserved. PP-EVZ-US-0861 February 2016

**Be among the first to get updates about EVZIO and receive a Trainer to show patients just how easy it is to use EVZIO.**

